

SAMPLING METHODS FORM

RegiSCAR: severe cutaneous adverse reactions

SAMPLE FOR RESEARCH PURPOSES

BIOLOGICAL SAMPLES

• Sterile blood

Type of sample	CASES				CONTROLS
	1 st sampling < day 5		2 nd sampling ** (4-8 weeks after notification)		
	Adults	Children	Adults	Children	
with EDTA	2 x 7 ml	1 x 7 ml *	-	-	2 x 7 ml
with heparin	2 x 7 ml	2 x 7 ml	2 x 7 ml	2 x 7 ml	-
without anticoagulant	1 x 7 ml	1 x 5 ml	1 x 7 ml	1 x 5 ml	-

* at least 5 ml

** except for AGEP

- **Blister fluid** : from patients with blisters all available fluid will be collected by puncture in a sterile plastic tube

LABELING

Interview number

Don't use labels allowing the donor identification

CONSERVATION

All samples : at room temperature

SHIPMENT

- At room temperature
- As soon as possible after sampling
- In insulated secured packages for diagnostic specimens conform to the IATA regulation
- Join the "Sample form" with all requested information (EN.T01.REC.07)
- By air courier: samples should reach CEPH within 48 hours of collection (in any case not more than 72 hours). They should arrive to CEPH before Friday 2 p.m.
- Notify the shipment and the air bill tracking number preferably by e-mail to crb@cephb.fr (or by fax to +33 1 53 72 51 28)

ADDRESS OF SHIPMENT :

Fondation Jean Dausset - CEPH
 Service de réception et traitement des échantillons
 27, rue Juliette Dodu
 75010 Paris - France

CONTACT :

Service of reception and treatment of biological samples

Tel. : +33 1 53 72 51 61 / 51 68

Fax : +33 1 53 72 51 28